

Exhibit A

# Loop Mask (Model – MN)



## Model-MN Face Mask

- 2 layers of 100% Polypropylene (Fluid Resistant)
- Breathable
- Soft inner surface
- Adjustable Nose Clip

<b>Donning Instructions</b>	<b>Technical Data</b>								
<ol style="list-style-type: none"><li>1. Hang loop of mask around neck with material resting in front of you.</li><li>2. Grab one strap with each hand and pull upward to snug the straps.</li><li>3. Flip the mask upwards covering your face.</li><li>4. Secure nose clip.</li></ol>	<table border="1"><tbody><tr><td data-bbox="800 1606 992 1682">Material</td><td data-bbox="992 1606 1461 1682">2 Layers 100% Polypropylene 1.25 oz/yd<sup>2</sup>.</td></tr><tr><td data-bbox="800 1682 992 1730">Seams</td><td data-bbox="992 1682 1461 1730">Tex-60 Polyester Thread</td></tr><tr><td data-bbox="800 1730 992 1778">Nose Clip</td><td data-bbox="992 1730 1461 1778">Metal Clip/Plastic Double Wire Tie</td></tr><tr><td data-bbox="800 1778 992 1869">Straps</td><td data-bbox="992 1778 1461 1869">Non-woven Spunbond Polypropylene</td></tr></tbody></table>	Material	2 Layers 100% Polypropylene 1.25 oz/yd <sup>2</sup> .	Seams	Tex-60 Polyester Thread	Nose Clip	Metal Clip/Plastic Double Wire Tie	Straps	Non-woven Spunbond Polypropylene
Material	2 Layers 100% Polypropylene 1.25 oz/yd <sup>2</sup> .								
Seams	Tex-60 Polyester Thread								
Nose Clip	Metal Clip/Plastic Double Wire Tie								
Straps	Non-woven Spunbond Polypropylene								

*This is not a respirator – this is intended to be used as a loop mask.*